




Cardinal[®]


DETECTO

SET-UP INSTRUCTIONS

The installation consists of merely connecting (A) the assembly column and head to (B) the platform base. Included in the carton are the required number of screws and lock washers for assembly.

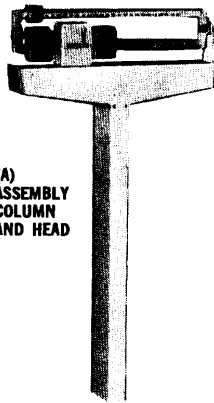

(A)
ASSEMBLY
COLUMN
AND HEAD

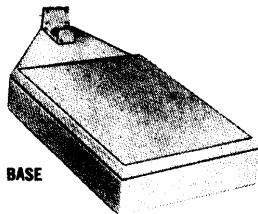





(B) PLATFORM BASE

STEP ONE
Carefully remove the scale from carton and remove all wrappings and fillers.

STEP TWO
Slip column over column bracket on platform base. Hold column straight so that steel yard rod enters cutout in base connector.

STEP THREE
Screw column to column bracket. (Two screws in front of column, four in back. Be sure to use enclosed lock washers.)

STEP FOUR
Lay down scale, with column horizontal to the floor (as illustrated). Keeping the steel yard rod hook out of the way, push lever loop upward as far as it will go. When the loop is in position, slip the steel yard hook into it.

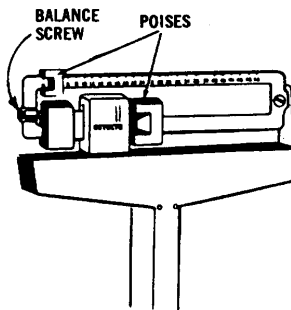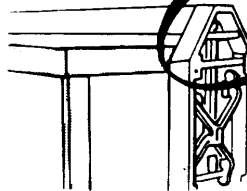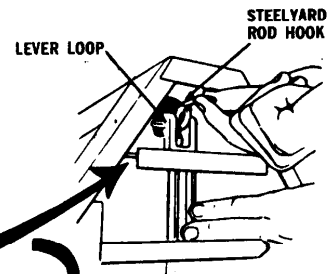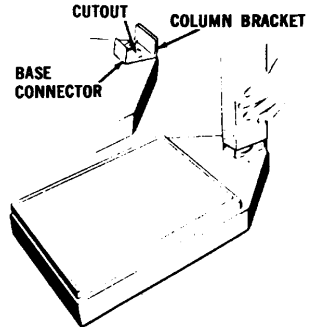

STEP FIVE
To balance scale move both poises to zero (0) and turn balance screw to right or left until scale balances.

FOR ACCURATE WEIGHING, SCALE MUST STAND LEVEL