




DLM
Digital Length Measuring
Device
User Manual

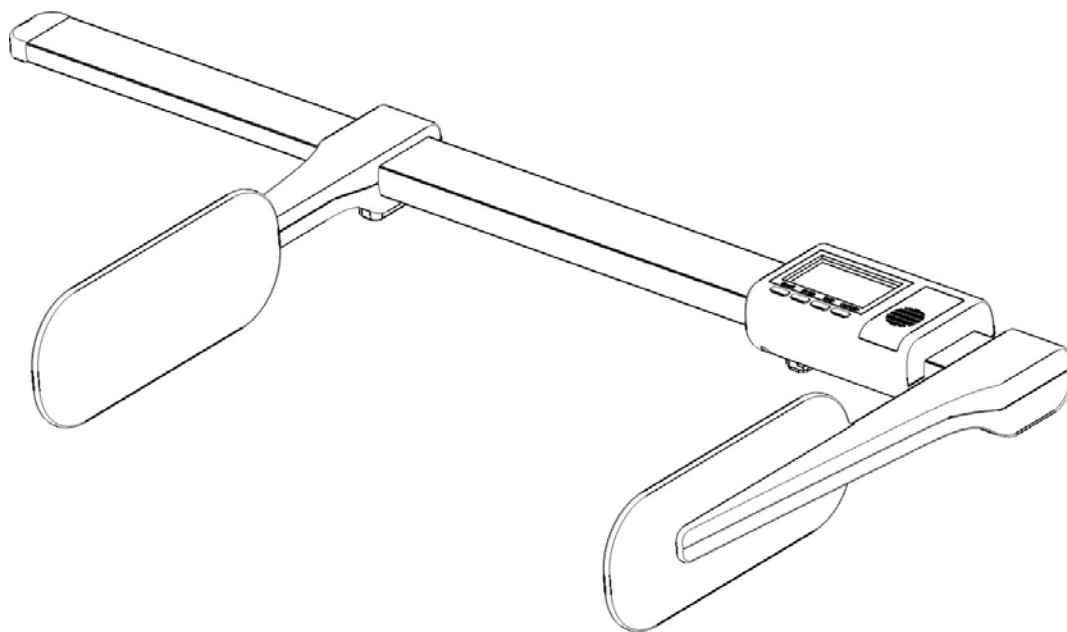


Cardinal Scale Mfg. Co.
203 E. Daugherty, Webb City, MO 64870 USA
8525-0367-O1 Rev A Ph: 417-673-4631 • Fax: 417-673-2153
09/17 www.detecto.com


Technical Support: Ph: 866-254-8261 • tech@cardet.com

ASSEMBLY


1

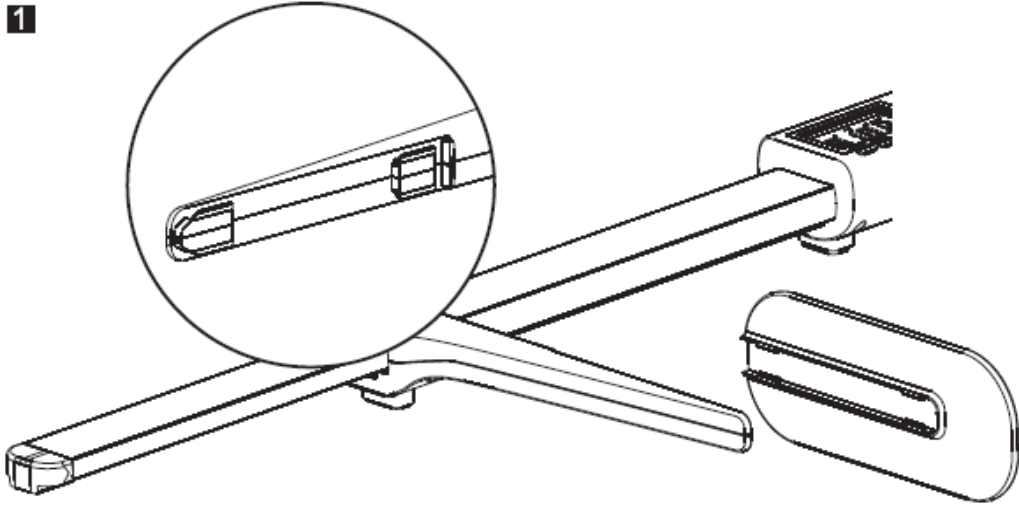

2



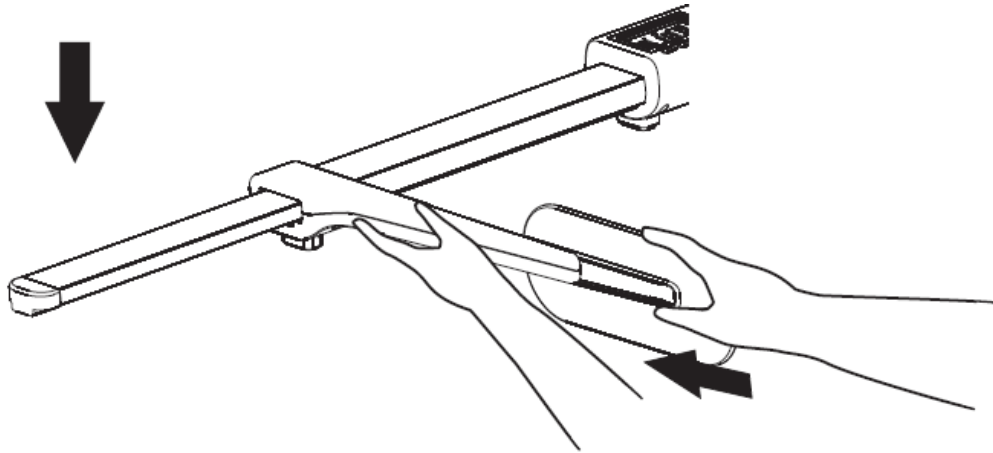


3

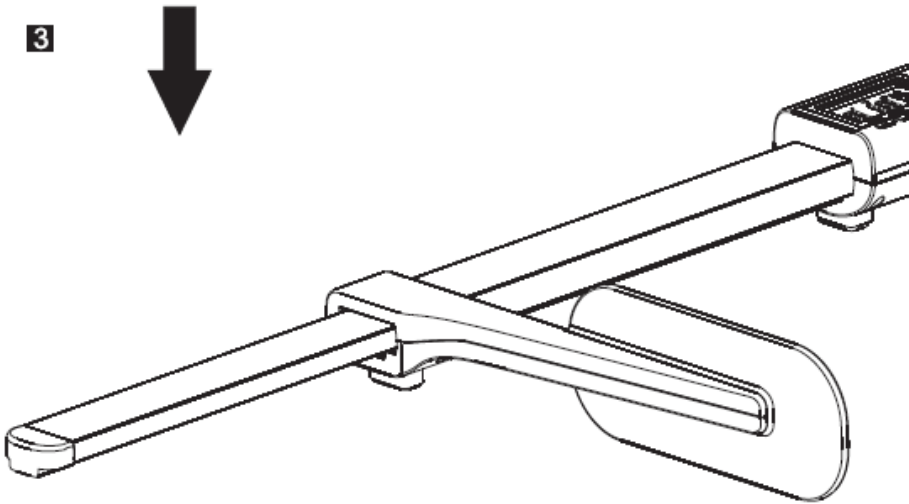




HOW TO MEASURE YOUR BABY

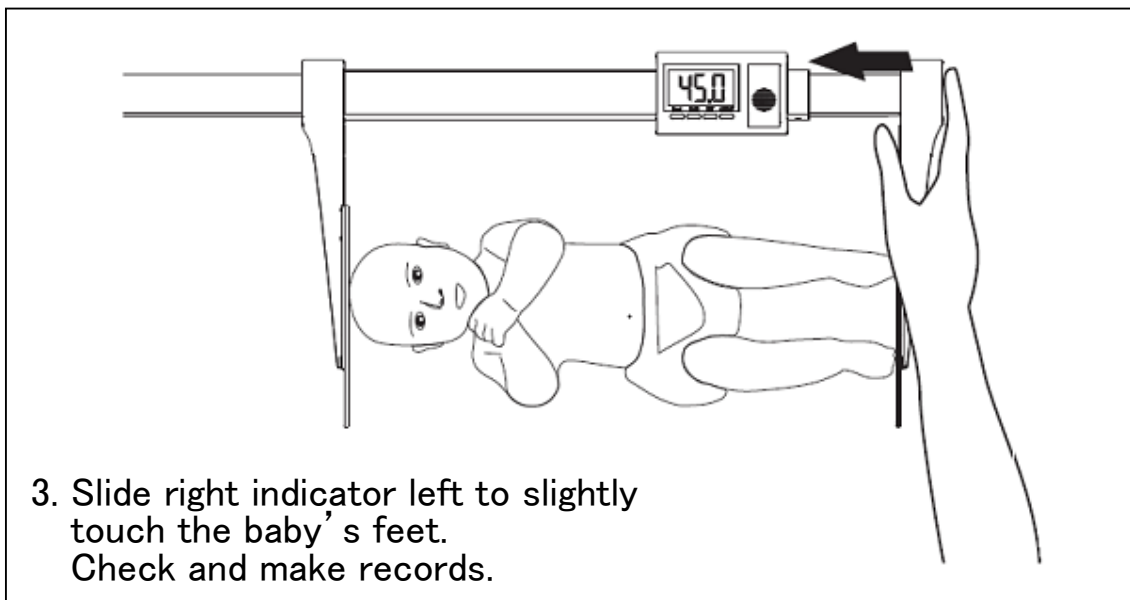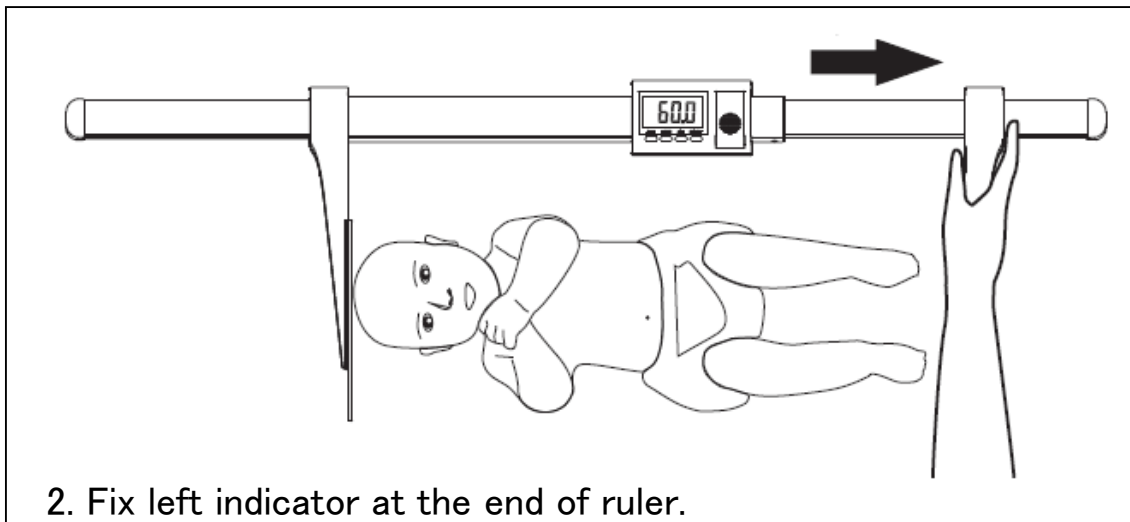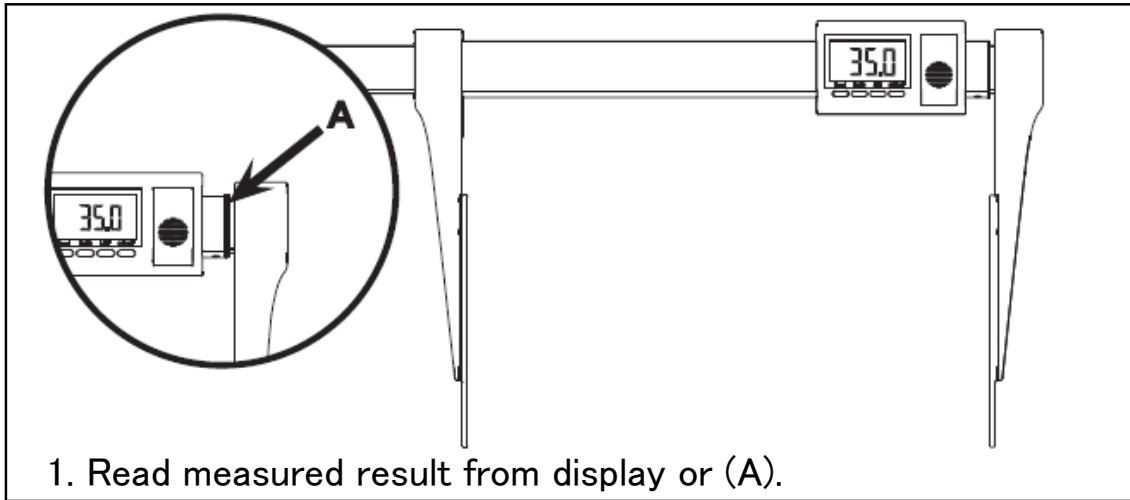


KEY FUNCTION

* To Calibrate The Height Measurement

1.

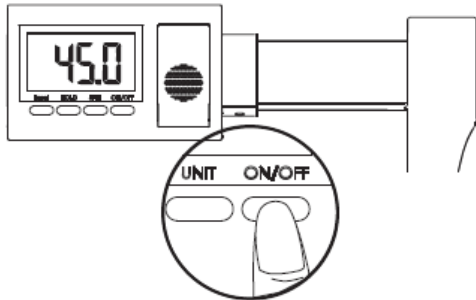

2.

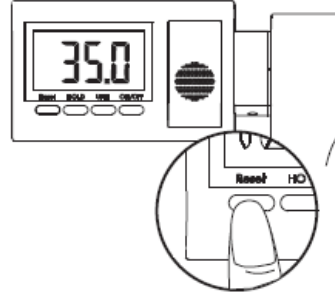

■ ON/OFF ■ UNIT ■ HOLD

■ ON/OFF

1. Press this key to power up the height rod; the display will show current height
2. Press the ON/OFF key to switch off the height rod.

■ UNIT

1. Press this key to select height unit (cm and ft/ inch and inch).

■ HOLD

1. Press to hold the reading of height of baby when right plate slightly touches baby's feet.
2. Press again to release the reading.

■ Reset

1. Reset function: slide down indicator to end and press Reset kg to zero the system.




Measurement range: 35-80cm / $13 \frac{3}{4}$ ~ $31 \frac{1}{2}$ inch

Graduations: 1mm. $\frac{1}{16}$ inch

Dimensions (W x H x L): 29 x 7 x 62 cm

Weight: 700g

Temperature range: +10°C up to +40°C

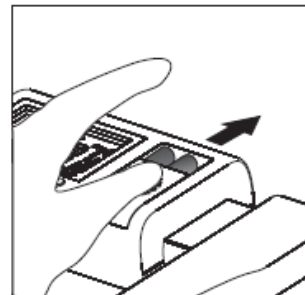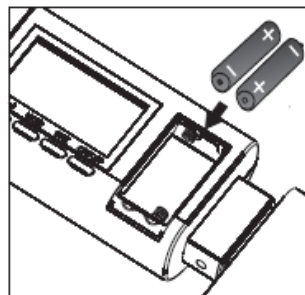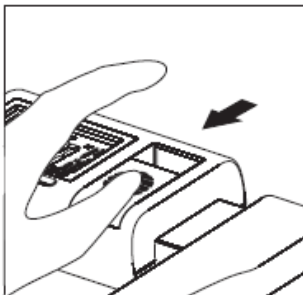

*Batteries installation: 1.5V AAA x 2