# DETECTO®

**Model MB130  
Baby Scale  
User Manual**

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## FCC COMPLIANCE STATEMENT

This equipment generates uses and can radiate radio frequency and, if not installed and used in accordance with the instruction manual, may cause interference to radio communications. It has been tested and found to comply with the limits for a Class A computing device pursuant to Subpart J of Part 15 of FCC rules, which are designed to provide reasonable protection against such interference when operated in a commercial environment. Operation of this equipment in a residential area may cause interference, in which case the user will be responsible for taking whatever measures necessary to correct the interference.

You may find the booklet "How to Identify and Resolve Radio-TV Interference Problems" prepared by the Federal Communications Commission helpful. It is available from the U.S. Government Printing Office, Washington, D.C. 20402. Request stock No. 001-000-00315-4.

## PROPER DISPOSAL

When this device reaches the end of its useful life, it must be appropriately disposed of. It must not be disposed of as unsorted municipal waste. Within the European Union, this device should be returned to the distributor from where it was purchased for proper disposal. This is in accordance with EU Directive 2002/96/EC. Within North America, the device should be disposed of in accordance with the local laws regarding the disposal of waste electrical and electronic equipment.

It is everyone's responsibility to help maintain the environment and reduce the effects of hazardous substances in electrical and electronic equipment on human health. Please do your part by ensuring that the device is correctly disposed of. The symbol shown to the right indicates that this device must not be disposed of in unsorted municipal waste programs.

## COPYRIGHT

All rights reserved. Reproduction or use, without expressed written permission, of editorial or pictorial content, in any manner is prohibited. No patent liability is assumed with respect to the use of the information contained herein.

## DISCLAIMER

While every precaution has been taken in the preparation of this manual, the Seller assumes no responsibility for errors or omissions. Neither is any liability assumed for damages resulting from the use of the information contained herein. All instructions and diagrams have been checked for accuracy and ease of application; however, success and safety in working with tools depend significantly upon the individual's accuracy, skill, and caution. For this reason, the Seller cannot guarantee the result of any procedure contained herein. Nor can they assume responsibility for any damage to property or injury to persons occasioned from the procedures. Persons engaging in the procedures do so entirely at their own risk.

<b>Serial Number</b> _____
<b>Date of Purchase</b> _____
<b>Purchased From</b> _____
_____
_____
_____
RETAIN THIS INFORMATION FOR FUTURE USE

## INTRODUCTION

The Model MB130 Baby Scale is a 44 lb (20 kg) capacity battery-operated scale. The scale is equipped with a removable baby tray to allow weighing small children. With a 9-volt alkaline battery (not included), the scale can be used as a portable scale, or with the included AC power adapter, the scale can be used in a fixed location.

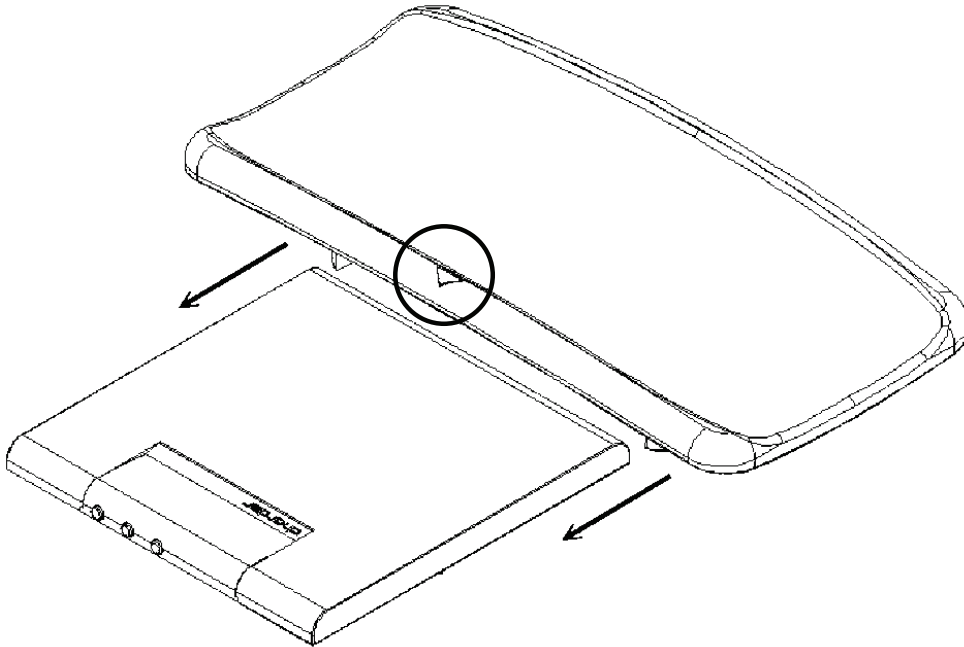
This manual contains information on the operation of this scale. Please read it before using the scale and keep it available for future reference.

## SPECIFICATIONS


<b>Capacity:</b>	0 to 22 lb x 0.01 lb (0 to 10 kg x 5 g) 22 to 44 lb x 0.02 lb (10 to 20 kg x 10 g)
<b>Tray:</b>	Removable to allow the scale to weigh small children
<b>Weight Units:</b>	lb/kg
<b>Function Keys:</b>	POWER/ZERO/TARE (⏻/→O/T←), UNIT, and HOLD/MILK INTAKE
<b>Stabilization Time:</b>	1 to 2 Seconds
<b>Operating Temperature:</b>	40° to 105°F (5° to 40°C)
<b>Display:</b>	1.0" (25 mm) LCD display with 5 active digits
<b>Power Requirements:</b>	9-volt battery (not included) or AC power adapter (included)
<b>Power Saving Feature:</b>	Auto Shut-off after 3 minutes
<b>Dimensions:</b>	
<b>Tray:</b>	22" L x 11.4" W x 2.56" H (560 x 290 x 65 mm)
<b>Scale:</b>	12.6" L x 12.2" W x 1.38" H / (320 x 310 x 35 mm)

## SCALE ASSEMBLY

1. Install tray from the rear of scale, sliding it to the front of scale (toward the display). Note that arrow on tray should point to front of scale.

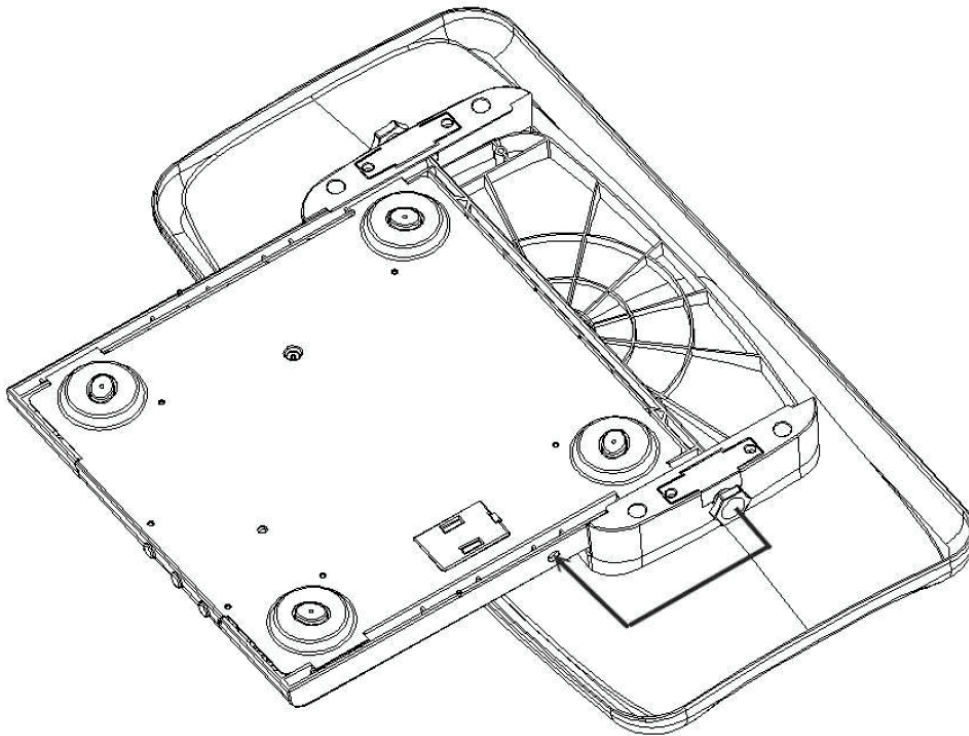
2. Turn scale upside down to screw tray and platform together properly.


## SCALE ASSEMBLY, CONT.

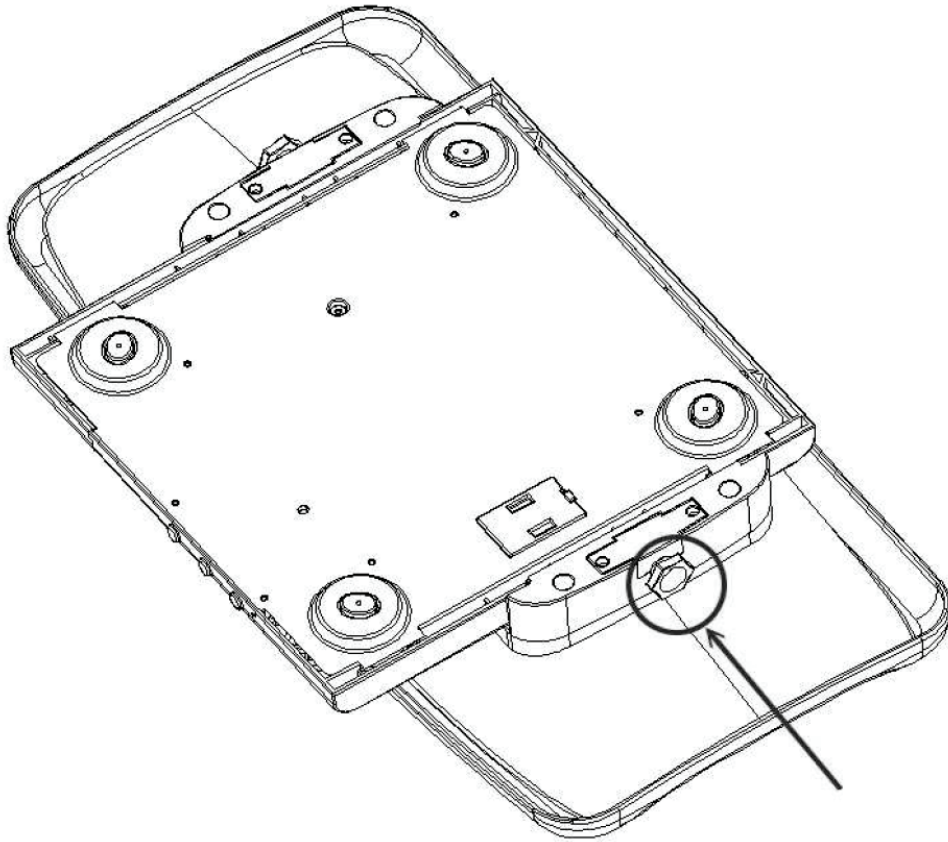3. Hand-tighten the screws to secure the tray to the platform

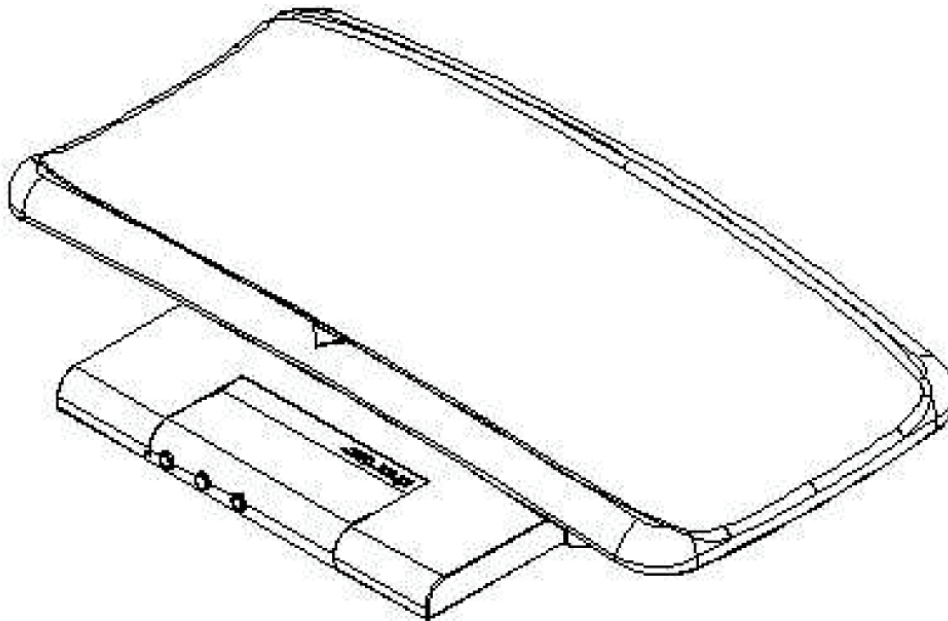
4. Scale assembly is finished. Turn scale over to begin weighing operations.


## AC POWER ADAPTOR

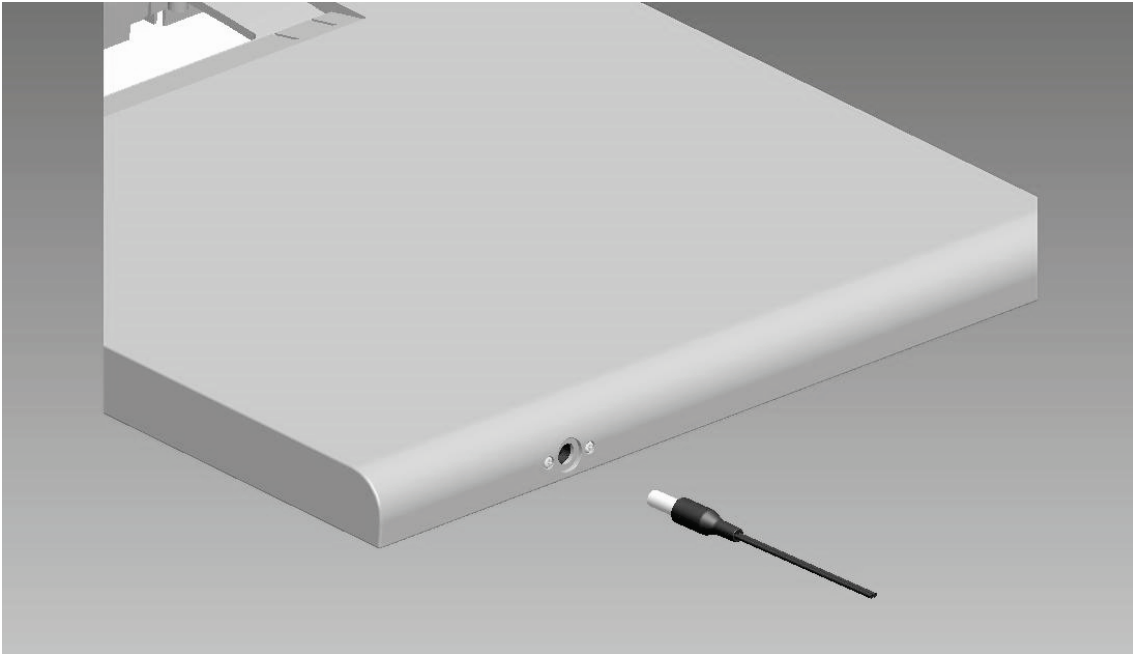1. Locate AC power adaptor jack at rear of scale.

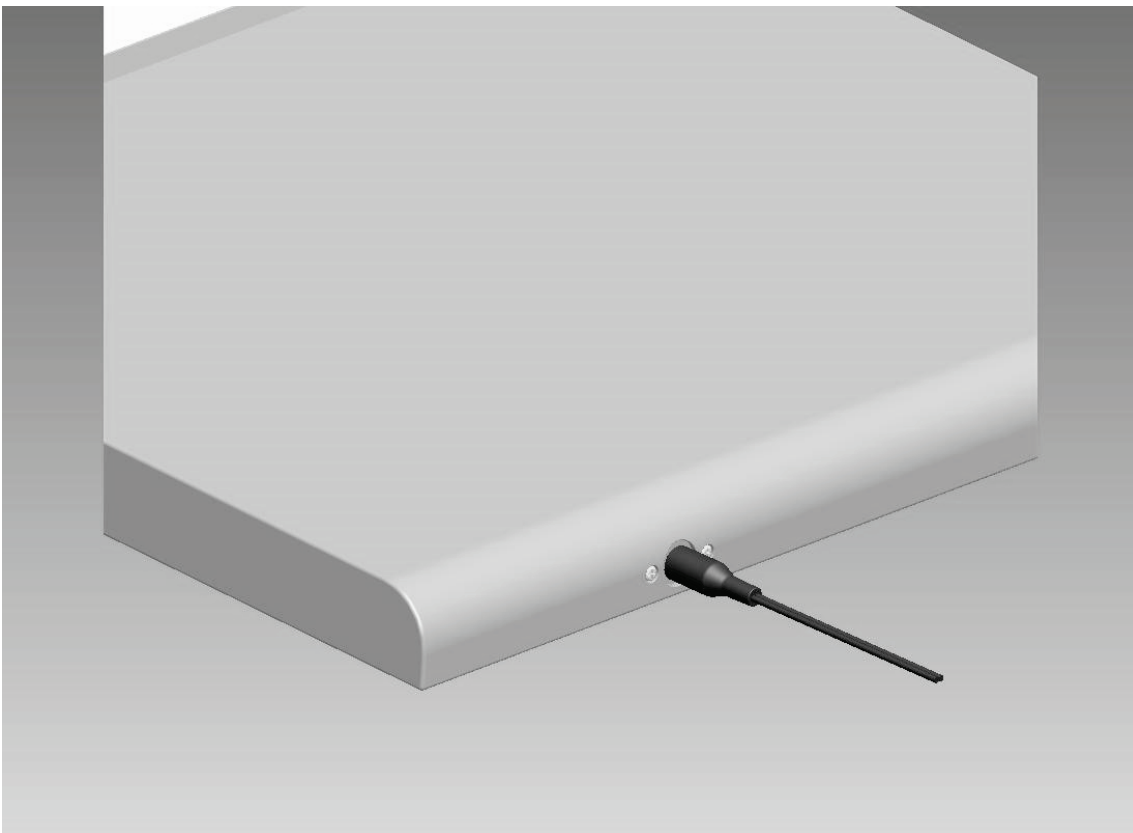
2. Install AC power adaptor into jack on rear of scale as shown below.


# INSTALLING/REPLACING THE BATTERY

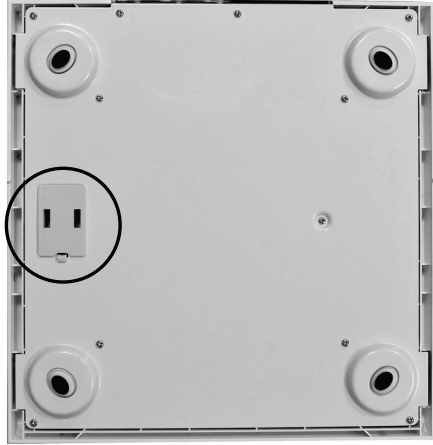1. Turn scale upside down to locate battery housing.

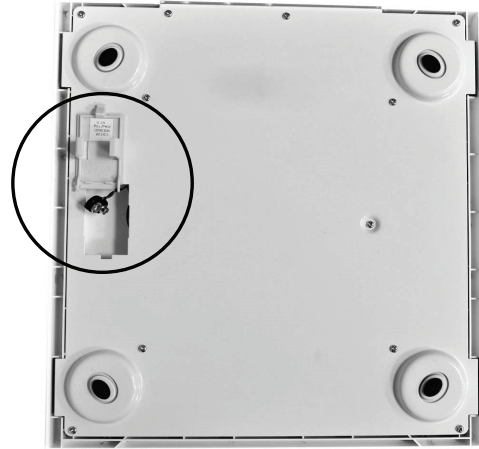
2. Remove battery housing from scale.

3. The scale uses a 9V alkaline battery.

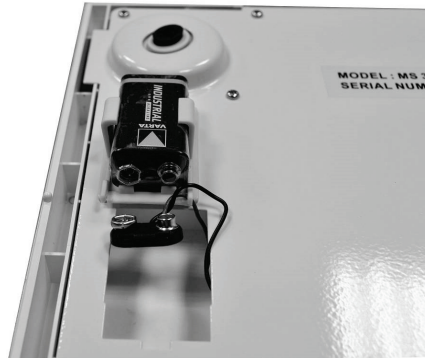
4. Install the 9V alkaline battery.

5. Replace battery housing in scale to complete procedure.

# KEY FUNCTIONS

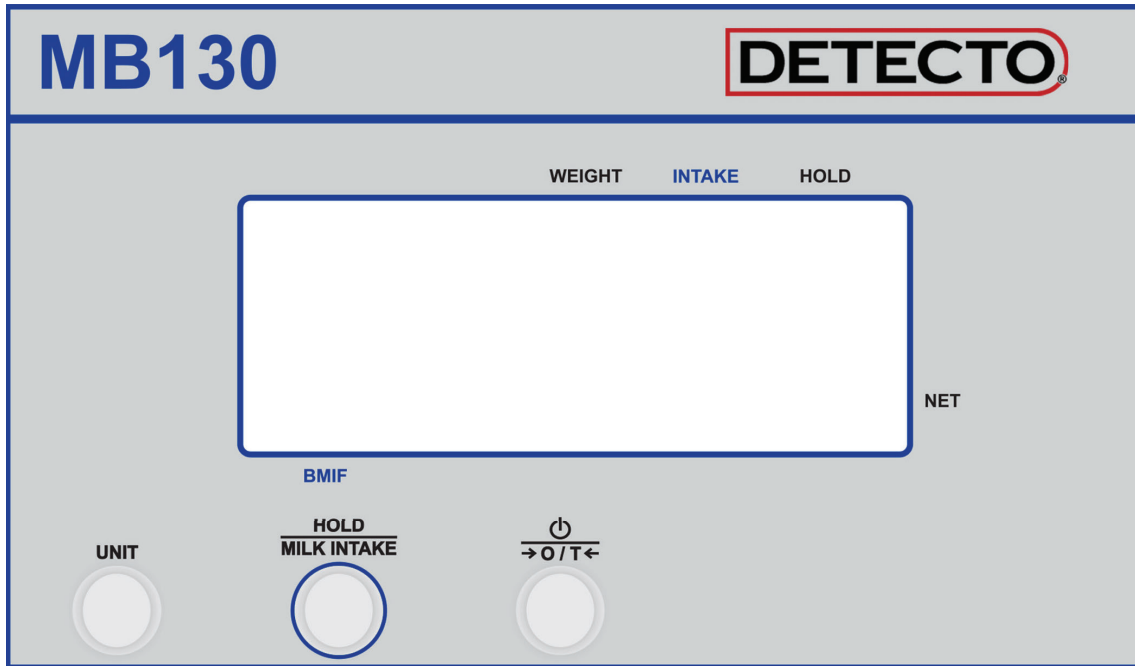**NEVER** leave an infant or child unattended while on the scale. Failure to maintain control of infant or child at all times while they are on the scale can result in serious injury to infant or child.

## KEYS

### ⏻/→0/T← (POWER/ZERO/TARE)

1. Press this key to turn the scale ON.
2. Press this key to zero the scale display before starting the weighing operation or to zero off (Tare) the weight of an item (i.e., blanket, pad, or toy) on the scale ( $\pm 2\%$  of full capacity).
3. Press this key and hold for 3 seconds to turn the scale OFF.

### HOLD/MILK INTAKE

1. Press this key to hold the weight value while weighing.
2. Press this key again to disable the weight hold function.
3. Press and hold until display changes to *StorE* to enable Breast Milk Intake Feature.
4. Press and hold until display exits BMIF function and returns to Zero in normal weighing mode.

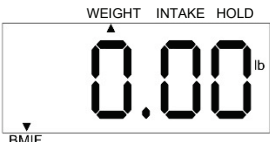
### UNIT

1. Press this key to change the weighing units from lb to kg or kg to lb.



## DISPLAY ANNUNCIATORS
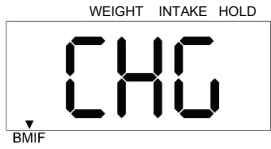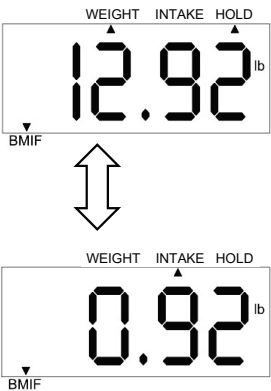
<b>WEIGHT</b>	The <b>WEIGHT</b> annunciator is located in the top center of the display. An arrow will be turned on to indicate the weight is at zero and the scale is ready to use.
<b>HOLD</b>	The <b>HOLD</b> annunciator is located at the top right of the display. An arrow will be turned on and <i>H o L d</i> be shown on the display for a few seconds to indicate that the weight from the last weighing operation is being held and the baby or child can be removed from the scale. This feature is useful if the baby or child is highly active.
<b>NET</b>	The <b>NET</b> annunciator is located in the right corner of the display. An arrow will be turned on to show that the weight displayed is net weight. Net weight is determined by zeroing the weight of an item from the scale such as blankets, pads, or toys. This ensures that only the baby's or child's weight is displayed.
<b>kg</b>	The <b>kg</b> annunciator is turned on to indicate the weight shown is in kilograms.
<b>lb</b>	The <b>lb</b> annunciator is turned on to indicate the weight shown is in pounds.
<b>BMIF</b>	An arrow will be turned on to indicate Breast Milk Intake Feature is enabled.
<b>INTAKE</b>	An arrow will be turned on to denote that the display is showing the Breast Milk Intake amount.

## BREAST MILK INTAKE FUNCTION

<p>Turn Scale On.</p>	
<p>Weigh baby before feeding.</p>	
<p>Press <b>MILK INTAKE</b> key until displays changes to <i>StorE</i>.</p>	
<p>BMIF annunciator will illuminate.</p>	
<p>Remove baby from scale.</p>	
<p>After feeding, place baby on scale.</p>	


## BREAST MILK INTAKE FUNCTION, CONT.

<p>Press <b>MILK INTAKE</b> key until displays changes to <i>CHG</i>.</p>	
<p>Baby may be removed from scale.</p> <p>Display will alternate between current baby weight and milk intake (difference from first weight).</p>	
<p>Press <b>MILK INTAKE</b> key until display exits BMIF function and returns to Zero in normal weighing mode.</p>	

# ERROR MESSAGES

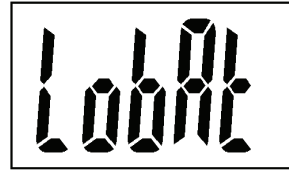
## Low Battery

This message indicates that the battery should be replaced soon. If the battery voltage drops too low, continued use with a low battery can affect weighing accuracy. Remove the old battery and replace it with a new battery for operation.


## Overload

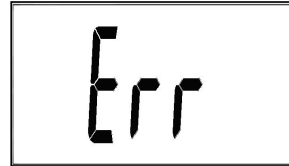The load is over the scale capacity. Reduce the load and try again. If the trouble still exists, please call your scale service representative.


## Over Zero

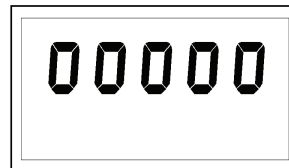The load is over the limit the scale can zero when powering on. Reduce the load and try again. If the trouble still exists, please call your scale service representative.

## Under Zero

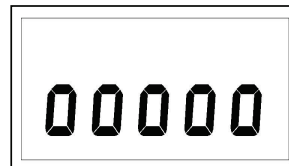The load is under the limit when powering on. Power OFF the scale and try again. If the trouble still exists, call your scale service representative.

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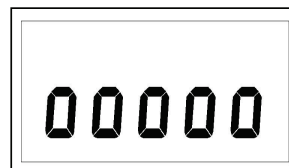
# STATEMENT OF LIMITED WARRANTY

DETECTO warrants its equipment to be free from defects in material and workmanship as follows: DETECTO warrants to the original purchaser only that it will repair or replace any part of equipment which is defective in material or workmanship for a period of **two (2) years from date of shipment**. Detecto shall be the sole judge of what constitutes a defect.

During the **first ninety (90) days** DETECTO may choose to replace the product at no charge to the buyer upon inspection of the returned item.

**After the first ninety (90) days**, upon inspection of the returned item, DETECTO will repair or replace it with a remanufactured product. The customer is responsible for paying for the freight both ways.

This warranty does not apply to peripheral equipment not manufactured by DETECTO; this equipment will be covered by certain manufacturer's warranty only.

This warranty does not include replacement of expendable or consumable parts. This does not apply to any item which has deteriorated or damaged due to wear, accident, misuse, abuse, improper line voltage, overloading, theft, lightning, fire, water or acts of God, or due to extended storage or exposure while in purchaser's possession. This warranty does not apply to maintenance service. Purchased parts will have a ninety (90) day repair or replacement warranty only.

DETECTO may require the product to be returned to the factory; item(s) must be properly packed and shipping charges prepaid. A return authorization number must be obtained for all returns and marked on the outside of all returned packages. DETECTO accepts no responsibility for items lost or damaged in transit.

## Conditions Which Void Limited Warranty

This warranty shall not apply to equipment which:

- A.) Has been tampered with, defaced, mishandled or has had repairs and modifications not authorized by DETECTO.
- B.) Has had serial number altered, defaced, or removed.
- C.) Has not been properly grounded according to Detecto's recommended procedure.

## Freight Carrier Damage

Claims for equipment damaged in transit must be referred to the freight carrier in accordance with freight carrier regulations.

This warranty sets forth the extent of our liability for breach of any warranty or deficiency in connection with the sale or use of the product. DETECTO will not be liable for consequential damages of any nature, including but not limited to, loss of profit, delays or expenses, whether based on tort or contract. Detecto reserves the right to incorporate improvements in material and design without notice and is not obligated to incorporate improvements in equipment previously manufactured.

The foregoing is in lieu of all other warranties, express or implied including any warranty that extends beyond the description of the product including any warranty of merchantability or fitness for a particular purpose. This warranty covers only those DETECTO products installed in the forty-eight (48) contiguous continental United States.

Ph. (800) 641-2008  
E-mail: [detecto@cardet.com](mailto:detecto@cardet.com)  
102 E. Daugherty  
Webb City, MO 64870

04/22/2024  
Printed in USA  
D268-WARRANTY-DET-B

*DETECTO*  
*102 E. Daugherty, Webb City, MO 64870 USA*  
*Ph: 417-673-4631 or 1-800-641-2008*  
*Fax: 417-673-2153*  
*www.Detecto.com*  
*Technical Support: 1-866-254-8261*  
*E-mail: tech@cardet.com*