



# Model DR150 Calibration Instructions

Before beginning calibration, the following equipment is required:

Calibrated test weights    60 kg or 150 lbs

## To begin calibration:

1. Make certain scale platform is empty.
2. With the scale off, press and hold the **ON/OFF** key.
3. Press **kg/lb** key three times and then release both keys.
4. Scale display will show *2000 ~ 9000* count.
5. Now press **kg/lb** key to select kg or lb to calibrate the scale.
6. Press **TARE** key.
7. When scale display shows *0.0*, place 60kg (or 150lb) weight on the platform.
8. Scale display will show *8000 ~ 14000* count.
9. Press **TARE** key.
10. Scale display will show *CAL* then *60* (or *150*).
11. The calibration is finished.
12. Remove test weights from scale platform.
13. Turn scale off and then back on.
14. Scale is now ready for operation.