






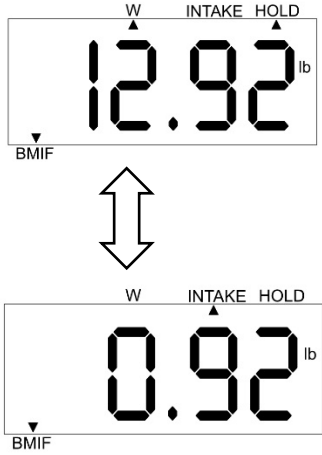


<p>Turn Scale On</p>	
<p>Weigh baby before feeding</p>	
<p>Press HOLD key until displays STORE</p>	
<p>BMIF Annunciator will illuminate</p>	
<p>Remove baby from scale.</p>	
<p>After feeding place baby on scale</p>	

<p>Press HOLD key until displays CHG</p>	
<p>Baby may be removed from scale Display will alternate between current baby weight and milk intake (difference from first weight)</p>	
<p>Press HOLD key until display exits BMIF function and returns to Zero in normal weighing mode.</p>	